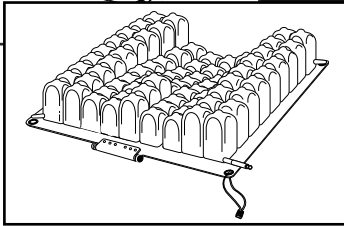
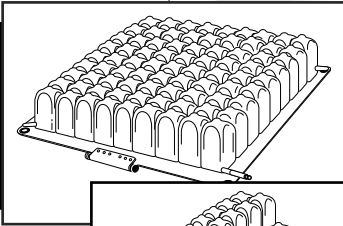
Deutsch

# SELECT

Cushioning Products  
Operations Manual

**QUADTRO® SELECT®**  
**CONTOUR SELECT™**

shape fitting technology®

QUADTRO® SELECT® and CONTOUR SELECT™ Cushions are manufactured in the U.S.A. by ROHO, Inc., 100 N. Florida Ave., Belleville, IL 62221-5429.

© The ROHO Group, Inc. 2003

The following are trademarks and registered trademarks of ROHO, Inc.:

ROHO®, AIR IN PLACE®, DRY FLOATATION®, ISOFLO™ Memory Control™, QUADTRO® SELECT®, CONTOUR SELECT™, Yellow Rope®, and shape fitting technology®.

**SUPPLIER:** This manual must be given to the user of this wheelchair cushion.

**USER:** Before using this wheelchair cushion, read this entire manual and save for future reference.

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100 North Florida Avenue  
Belleville, IL 62221-5429, U.S.A.

[www.therohogroup.com](http://www.therohogroup.com)

U.S.A. und Kanada: 1-800-851-3449 Fax 1-888-551-3449

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## ROHO, Inc. 3-Year Limited Warranty

### What is covered?

ROHO, Inc. covers any defects in materials or workmanship of the SELECT Cushioning Products (QUADTRO SELECT and CONTOUR SELECT only).

### For how long?

Three years from the date the SELECT Cushioning Product was originally purchased as a new cushion, no matter who is the present owner.

### What we will do.

Within a reasonable amount of time after you return the cushioning product to us, we will repair any defect in material and workmanship and ship it back to you, all free of charge within the warranty period.

### What we will not do.

We will not repair a cushion free of charge if it has been misused or damaged by an accident. This also includes damages stemming from "acts of God", eg., flood, tornado, earthquake, or other catastrophic event, eg., fire.

### What the customer must do.

The customer must send the SELECT Cushioning Product to us with the original sales ticket or other evidence of date of purchase, and he / she must pay all delivery costs. It must be sent to ROHO, Inc., 100 N. Florida Ave., Belleville, IL 62221-5429. We will not require the original sales ticket if you or the previous owner, if applicable, have completed and returned the warranty registration card, attached to this brochure, and we have it on file. Failure to submit the original sales ticket or failure to have the warranty registration card in our file voids this warranty.

### Any other questions.

Call us toll free at 1-800-851-3449 or 1-618-277-9150.

### Disclaimer

Any implied warranties, including those of MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE are also limited to three (3) years from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time. ROHO, Inc. shall not be responsible for any incidental or consequential damages with respect to economic loss or injuries to property, whether as a result of a breach of express or implied warranties, or otherwise. ROHO, Inc. will not be responsible for the results of careless handling, unreasonable or improper use of this product.

### Your State Law Rights and the Effect of this Warranty Upon Them

This warranty restricts the duration of all implied warranties to three (3) years and limits or excludes recovery for incidental or consequential damages. Some states do not allow limitations on how long implied warranties will last and some states do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from state to state.

### European Union Authorized Representative (MDD 93/42/EEC)

Medical Device Service GmbH (MDSS)  
Burkhardtstr.1  
D-30163 Hannover  
Germany

## REGISTRATION CARD 3-Year Limited Warranty

PLEASE COMPLETE, DETACH AND  
MAIL FOR WARRANTY REGISTRATION

Name \_\_\_\_\_

Street No. \_\_\_\_\_

City, State \_\_\_\_\_

Zip & Country \_\_\_\_\_

Area Code & Telephone (    ) \_\_\_\_\_

Email Address \_\_\_\_\_

User's Age?    18 and under    19-34    35-54    55-65    Over 65

### PRODUCT INFORMATION

Purchased From \_\_\_\_\_

Date Purchased \_\_\_\_\_

Is this your first ROHO cushion?    Yes    No

Serial Number of cushion \_\_\_\_\_  
(Printed on back of cushion along lip)

\*Type of Product:    QUADTRO® SELECT®    CONTOUR SELECT™

\*The 3-year limited warranty is only offered on the above SELECT Cushioning Products.

1. How did you learn about  
DRY FLOATATION® cushions?

- therapist    friend  
 doctor    advertisement  
 dealer

3. How was your ROHO® cushion  
paid for?

- insurance    Medicare  
 self/family  
 Other \_\_\_\_\_  
(Please specify)

2. Why the ROHO system?

- prescribed  
 suggested by a ROHO product user  
 unsatisfied with other brands

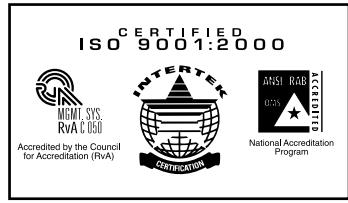
4. How many cushions do  
you currently own?

- 1    3    more than 4  
 2    4



\_\_\_\_\_  
signature

By signing this card, you agree that The ROHO  
Group may contact you in the future with  
additional product or promotional support.

LITG01010 \_ \_ \_ 10000 9/03

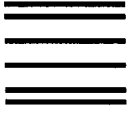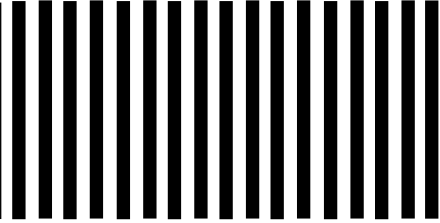NO POSTAGE  
NECESSARY  
IF MAILED  
IN THE  
UNITED STATES

**BUSINESS REPLY MAIL**  
FIRST CLASS PERMIT NO. 846 BELLEVILLE, ILLINOIS, U.S.A.

POSTAGE WILL BE PAID BY ADDRESSEE

100 N. Florida Avenue  
Belleville, Illinois 62221-5429 U.S.A.