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## 1 General instructions

### 1.1 Introduction

These operating instructions contain information about how to use the shower stool safely and properly.

Read the operating instructions thoroughly prior to the use of the product and follow all instructions.

Keep all documents in a safe place.

### 1.2 Proper use

The shower stool is used solely and exclusively as an aid to shower adolescents and adults in the bathroom. It allows persons with reduced mobility to shower sitting down. Any other form of use is not permitted.

The shower stool may not be used by children. The shower stool is not suitable for persons who are smaller than 146 cm or weigh less than 40 kg (BMI < 17).

#### Contraindications

No contraindications are known if the product is used properly.

### 1.3 Disposal

The product consists of recyclable materials.

Old appliances and packaging must be disposed of and recycled in accordance with the valid national regulations.

### 1.4 CE mark

The product complies with the medical products directive (EU) 2017/745.

### 1.5 Reuse

The product is suitable for reuse. The amount of reuse depends on the condition of the product in question.

Prior to reuse, follow the manufacturer's reconditioning and hygiene guidelines, and check the technical and functional safety of the product as well.

### 1.6 Service life

The product is designed for a service life of at least 3 years, provided that it is used in accordance with its intended use and that all cleaning and safety instructions are observed.

This service life can be exceeded if the product is carefully handled, cared for, and used.

## 2 Safety instructions

Do not overload the shower stool (max. user weight: 130 kg).

Position shower stool only on level surface.

Do not use shower stool as a climbing assistance nor stand on it.

Do not load the shower stool one-sidedly.

All chair legs must be adjusted to the same height.

Check shower stool for damage prior to every use. Do not use a damaged shower stool.

In case of severely reduced mobility, use the shower stool with the help of an assistant.

Hot water, sunlight or hot air can cause the surface of the product to become hot. Risk of burns!

Serious incidents relating to the product must be reported to the manufacturer and the competent authority.

## 3 Description

The product is available in the following versions:

- Shower stool (Fig. 1)
- Shower stool with rotary seat (Fig. 2)

The shower stool consists of a seat section and three chair legs. The ends of the chair legs are provided with large rubber pads that enhance anti-slip safety.

### 3.1 Technical data

Refer to "Technical data" at the end of the operating instructions.

## 3.2 Label

The label (Fig. 4) is attached to the bottom side of the seat section and contains the following information (exemplary illustration):

Sales or service address (1, Fig. 4), conformity marking (2, Fig. 4), note (observe operating instructions) (3, Fig. 4), maximum user weight (4, Fig. 4), period of production (5, Fig. 4), product identification number (6, Fig. 4), product name (7, Fig. 4), marking as a medical product (8, Fig. 4), batch number (9, Fig. 4), product type (10, Fig. 4), barcode (11, Fig. 4), item number (12, Fig. 4) and manufacturer (13, Fig. 4).

## 4 Use

**CAUTION** Do not allow any oily or strongly soapy substances to get to the rubber pads. Otherwise, the shower stool may slip.

**CAUTION** When using the shower stool, keep a still position and avoid excessive body movements. Otherwise, there is a risk of slipping.

1. Place the shower stool in the shower.
2. Sit down on the dry seat section.

## 5 Assembly

### 5.1 Attachment of chair legs

**CAUTION** Risk of crushing while fitting the chair legs. Fit the chair legs carefully.

1. Pull clip (2, Fig. 3) out of the chair leg.
2. Push chair leg (3, Fig. 3) as far as it will go onto the connecting piece (1, Fig. 3) of the seat section. When doing this, the broad side of the rubber pad (6, Fig. 3) must point inwards.
3. Push clip (2, Fig. 3) through the hole in the chair leg and the connecting piece until it snaps in.
4. Pull clip (4, Fig. 3) out of the chair leg and the extendable section.
5. Pull extendable section (5, Fig. 3) out to the desired seat height.
6. Push clip (4, Fig. 3) through the hole in the chair leg and the extendable section until it snaps in.

## 6 Disassembly

Pull clip out and remove chair leg.

For the model with rotary seat: refer to the separate instructions for removal of the rotary seat and the seat cover on the rotary seat.

## 7 Maintenance and care

### 7.1 Maintenance

When adhering to the specified care and safety instructions, the product is maintenance-free.

### 7.2 Care

**NOTICE** Do not use any scouring agent. Otherwise, the plastic surface may be damaged.

The product can be cleaned with lukewarm water mixed with a mild detergent (without solvents and of a pH value of 5-9). Care for the plastic piece with an off-the-shelf plastic cleaner.

If required, the shower stool can be disassembled to clean/disinfect the parts individually.

If necessary, or when using the product for more than one person, the stool surface can be disinfected with an off-the-shelf disinfectant (max. 70 % alcohol) **after cleaning**. Depending on the disinfectant used, the disinfectant will be sprayed on or applied with a cloth. Follow the disinfectant or detergent manufacturer's instructions.

The seat cover of the rotary seat can be removed and washed at 40 °C in a washing machine.

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