OLYMPOS ATR

THE ALL TERRAIN ROLLATOR

NO	Bruksanvisning	3–7
EN	User manual	8–12
DE	Bedienungsanleitung	13–17
FR	Manuel d'instructions	18–22
NL	Gebruiksaanwijzing	23–27

www.topromobility.com

Made in Norway

A 815270 (short – 67 cm)
815272 (long – 75 cm)

B 815212 only for size M

C 814059 only for size M

D 814046

E 814042

F 814040

G 814043


H 815260


I 814024

J 814023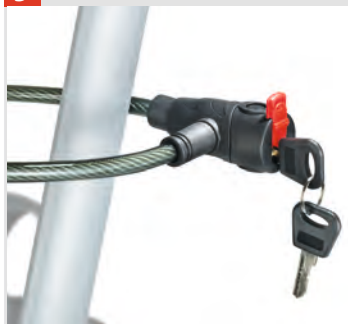


K 814068

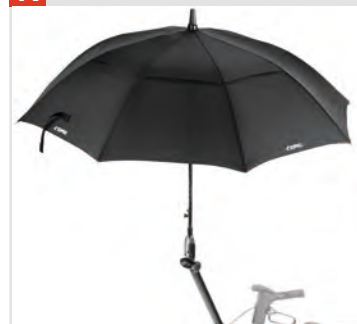
L 814038

M 815304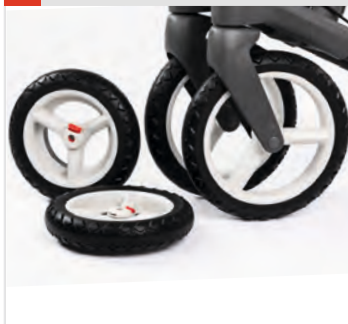

N 815332

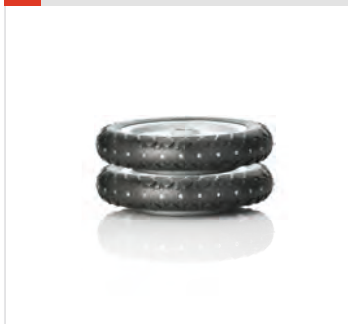O 814029 M 814035 L

P 814026

Q 814014












R 814009

TOPRO
OLYMPUS ATR

TOPRO OLYMPUS ATR M 814300
TOPRO OLYMPUS ATR S 814305

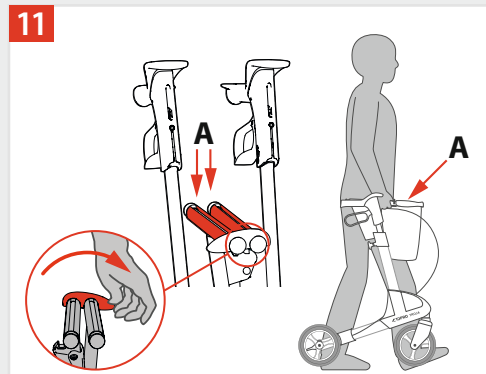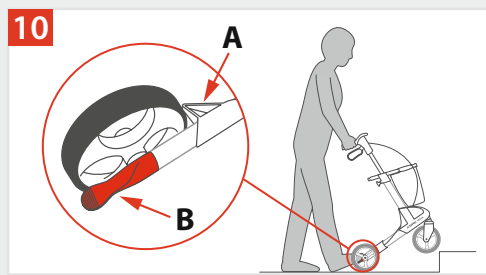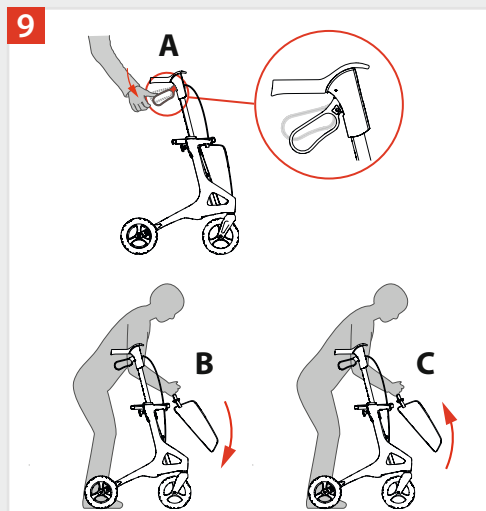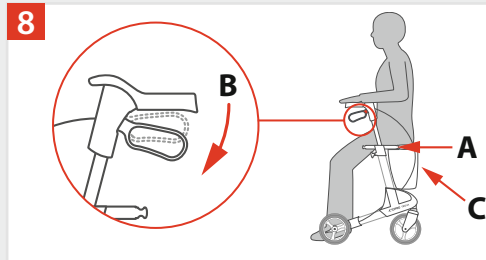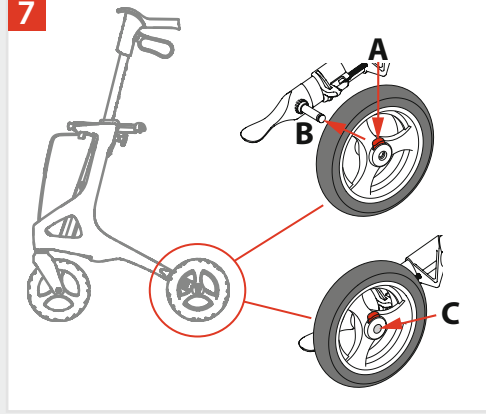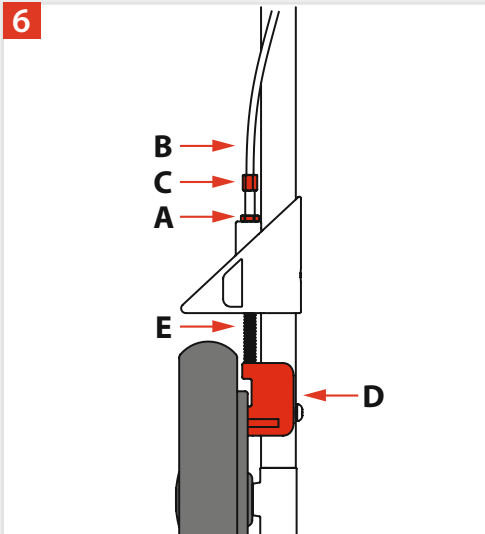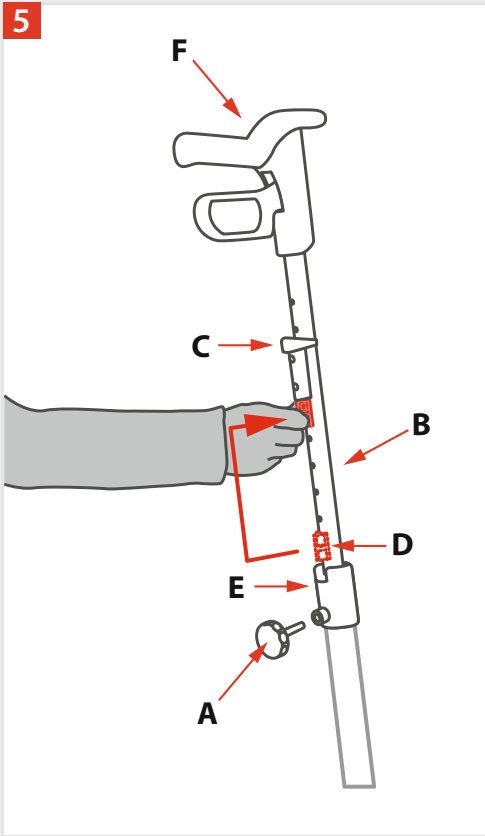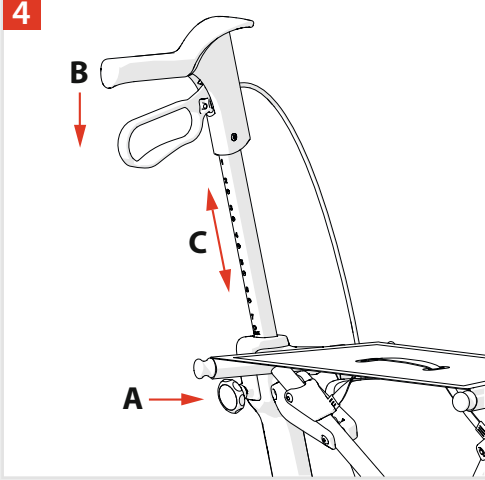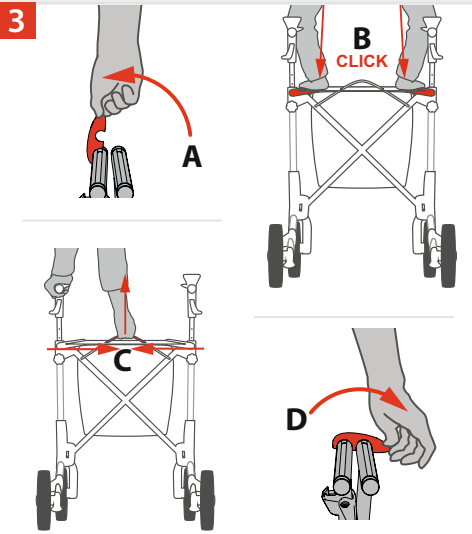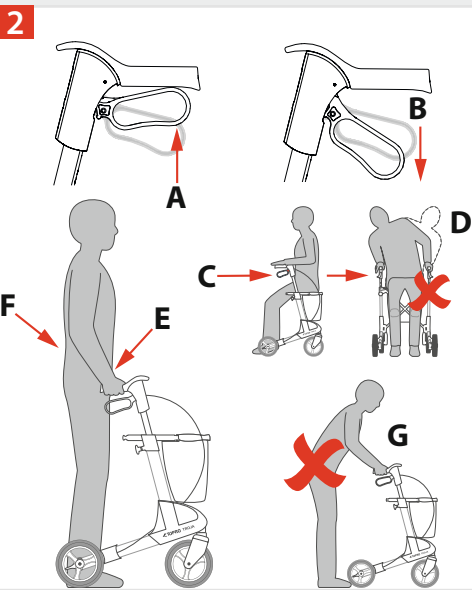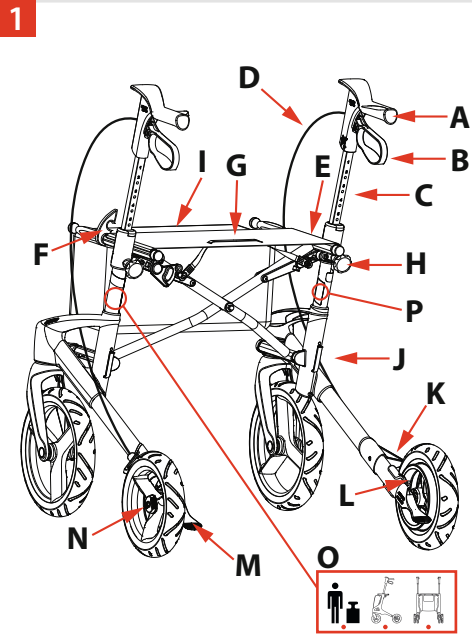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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and made in Norway. Advanced manufacturing equipment, quality and environmental security give great accuracy and maximum quality in all the levels of production. The product is declared to be in conformity with Medical Device Directive 93/42/EEC. It is tested and approved according to EN ISO 11199-2:2005.

On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topromobility.com.

Warranty // Expected lifespan of the product

TOPRO OLYMPOS ATR (All Terrain Rollator) is guaranteed free for faults and defects for 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake wire, wheel, bag, seat and grip) are exempt from the 7-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts and accessories have been or are being used on the product.

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

■ The red numbered squares below refer to the numbered illustrations on the inside of the cover.

Get to know your TOPRO PRODUCT FIG. 1

- | | | |
|-------------------------|--------------------------------------|--|
| A Handlebar | E Seat bars/carrying handle | I Shopping bag with lid and reflective band |
| B Brake handle | F Lock clamp | J Side frame |
| C Handlebar tube | G Seat with strap for folding | K Edge guard |
| D Brake wire | H Lock wheel for the handle | L Brake block |
| | | M Tilt function |
- N** Detachable rear wheels with TOPRO Quick Release
- O** The label on the rear side of the left tube shows the product name/model and
- Maximum weight of user
 - Maximum width of rollator
 - Maximum height of rollator
- P** A separate label on the right tube indicates the serial number (SN) and production year and month.

Measurements	TOPRO OLYMPOS ATR M	TOPRO OLYMPOS ATR S
Recommended user height	150–195 cm	140–170 cm
Maximum weight of user	150 kg	150 kg
Max. capacity shopping bag	15 kg /12 l	15 kg / 10 l
Handle height	785–945 mm	705–865 mm
Maximum height	980 mm	900 mm
Maximum width	680 mm	680 mm
Maximum length	770 mm	770 mm
Width between the handles	490 mm	500 mm
Seat width	470 mm	470 mm
Seat height	610 mm	545 mm
Turning diameter	970 mm	970 mm
Diameter front wheels	280 mm	280 mm
Diameter rear wheels	230 mm	230 mm
Width on front/rear wheels	42 mm	42 mm
Handles	Ergonomic – several holding positions (TOPRO Ergo Grip)	
Grip distance driving brake	70 mm	70 mm
Height when folded up	875 mm	850 mm
Width when folded up	270 mm	270 mm
Length when folded up	770 mm	770 mm
Product weight (including shopping bag)	8.9 kg	8.7 kg

Intention

The TOPRO OLYMPOS ATR shall give support to users with reduced balance and/or reduced walking ability. It is designed to be pushed, not pulled. The integrated seat provides an opportunity for the user to rest if

needed. The shopping bag is designed for carrying items weighing up to a total of 15 kg. The intended user is an adult. Height and weight restrictions apply, see table above.

Contraindications: The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities.

The product is designed and approved for outdoor use but can also be used indoors. We advise caution when using the OLYMPOS ATR indoors, as the maximum width of the rollator is 680 mm. The rollator is intended to be an aid for walking on roads and paths. It is not suitable in rough and/or steep terrain, neither for stairs.

Important to know about safety before use

- When folding and unfolding, be aware so you do not hurt your fingers.
- Be aware careful not to put your fingers in open holes or between stationary parts.
- Check the brakes before use every time, and that the rollator locks in the unfolded position.
- The rollator is not to be used by persons heavier than the user weight (see table above).
- Do not use to transport heavy loads or persons.
- The rollator is not a toy and therefore should be treated in an appropriate manner.
- Make sure that the lock wheel for height adjustment for the handles is locked tight.
- Be aware that some parts of the rollator might feel hot or cold when exposed to extreme temperatures.
- Training by a skilled person is advised, to get to know the device and to learn how to use it correctly.
- Do not alter the product as this might put your safety at risk and the guarantee will become void.

Important to know about safety when in use FIG. 2

- When stationary or parked, always use the parking brakes on both sides **B**.
- Before sitting on the seat **C**, make sure the parking brakes on both sides are in the locked position **B**.
- To avoid the risk of falling, do not lean or bend sideways or over the back of the rollator whilst in the seated position **D**. Do not tilt the rollator more than max. 5° sideways. For the same reason bags etc. must never be hung on the handlebars.
- When using the back support (accessory for additional comfort): to avoid risk of falling, do not lean too far backwards on the backrest.
- The driving brakes can be held in a little **A** to reduce speed but must not be in the locked position when the rollator is in motion.
- To give the right support and safety, make sure the rollator is kept as close to your body as possible **E**. Walk between the rear wheels, not behind them **G**.
- Walk straight, look forward and hold your backside in **F**. This is the easiest and safest way to keep your balance.
- Do not push the rollator too far from your body **G** as this can cause unnecessary strain and you could fall.
- Be aware that the rollator may roll faster than you when walking downhill, which may put you at risk of falling.
- Do not use the rollator on stairs. Be careful when descending kerbs.
- Do not use the rollator on slippery or icy surfaces.
- Be careful when wearing flared skirts and trousers as they could get caught in the wheels, which could lead to falling. We therefore recommend not wearing flared/wide clothing.
- For safety reasons it is prohibited to sit on the rollator during public transportation. Store your rollator securely and use the parking brake.
- Keep the rollator away from fire and hot items.

Assembling / adjusting

TOPRO OLYMPOS ATR is delivered assembled. Before use, you must unfold the rollator. The height of the handlebar tubes will then need adjusting to suit the user's needs. Please read the following paragraphs carefully.

Unfolding / folding FIG. 3

Unfolding: Release the lock clamp positioned at the seat bars **A**. Press the seat bars down and out **B**. The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The rollator is now locked in its open position.

Folding: Pull the strap on the seat **C** and press the side frames together. Lock the clamp **D** over the seat bars.

Lowering and height adjusting the handles FIG. 4

Lowering: Unscrew the lock wheel **A** (approx. 2 turns). Push the handlebar tubes down until they stop **B**. Tighten the lock wheel **A** again.

Height adjusting: Unscrew the lock wheel **A** (approx. 2 turns). Adjust the handlebar tubes to a suitable height then pull them slightly up and down **C** until you hear a clicking sound. Tighten the lock wheel **A** again. Adjust both handlebars to the same height by comparing the number scale on each side. *It is recommended that the handlebars are set to the same height as your wrists when your arms are hanging alongside your body.*

Height adjustment using the TOPRO Memory Function FIG. 5

With the tool free TOPRO Memory Function you have the facility to set the height of the handles at a standard setting to suit your requirements. This will enable you to easily find the same height again. This function may be necessary when the rollator is being transported or stored in as compact a manner as possible. Decide on the height of the handlebars (see number scale). Count visible holes at the back of the handlebar tubes. Unscrew the lock wheel **A**. Pull the tube **B** out of the frame. The fastening piece **C** will now follow the handlebar tube. Move the fastening piece **C** on the tube **B** so the number of holes corresponds to the number counted earlier. Fit the red memory chip **D** on the handlebar tube in the first available hole under the fastening piece **C**. Push the handlebar tube **B** down and into the side frame **E** until the fastening piece is locked. Screw the lock wheel **A** half in. Pull the handlebars **F** up until they stop, and then tighten the lock wheel **A** properly. From now on, when the handlebars are in the lowest position, they will stop in the pre-set position automatically when pulled up.

TOPRO Ergo Grip

TOPRO Ergo Grip offers ergonomical grip and support surfaces.

It combines an original hand grip position with a hand grip at the front of the handle. This gives you a better gripping position when getting up from a chair or couch. Always make sure that the parking brake is locked in when you do this. The new hand grip also offers extra support while standing. Several support options make it easy to vary your position when standing, thereby increasing your comfort and safety.

When pushing the rollator, your hands should always be kept in the original grip position in order to make the best possible use of the brakes.

Adjusting / checking / maintaining the brakes FIG. 6

Adjusting / checking: Loosen the nut **A**. Tighten the brake wire **B** with the adjustment screw **C**. Adjust the brake block **D** as close as possible to the inner drum brake, and then tighten the nut **A**. Turn the wheel around to check that the brake block is not touching the wheel. **Maintaining:** Clean the brake block **D** and the brake spring **E** frequently. The wheels and brakes can be replaced if they are worn out or damaged.

Using the brakes FIG. 2

Make sure the brakes are working efficiently before each trip. Each brake handle **A** works on each of the rear wheels. Wrong use can cause unsafe situations and damage to the rollator: *The rollator must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the brakes.* This indicates incorrect use and can cause wear and tear, impairing tyres and brake quality. **Parking brakes:** Push both brake handles **B** down until they lock. Both rear wheels are now locked. Squeeze both brakes handles upwards to release the parking brakes. **Driving brakes:** Pull both brake handles **A** upwards to reduce speed. *Do not use continuously while the rollator is in motion.*

Replaceable rear wheels with TOPRO Quick Release FIG. 7

If you want to remove the rear wheels for transportation or replacement, follow these instructions:

Removal: Loosen the brake lever. Push down the red button on the hub **A** and slide the wheel off the axle **B** while still pushing the button down.

Attachment: Loosen the brake lever. Slide the wheel onto the axle **B**. Push the wheel as far as possible inwards on the axle to make the wheel snap onto the axle. You will hear a "click" when the wheel is in place. The wheel is securely fastened when the axle is completely visible in the middle of the hub **C**. Gently pull the wheel to check that it is securely fastened to the axle.

After the new wheel has been applied to the rollator, make sure that the brake lever is reattached, and the brakes are functioning properly. Readjust them if necessary (see section Adjusting / checking / maintaining the brakes).

Seat FIG. 8

The rollator has a safe and convenient net seat, through which you have a view to the ground while walking. It can easily be wiped with a cloth if needed. The parking brakes must be on before using the seat **A**. Push both brake handles downwards until they lock with a “click” sound **B**. Both rear wheels are now locked. Sit with your back towards the shopping bag **C**. The maximum user weight is 150 kg.

Shopping bag FIG. 9

The shopping bag is enclosed. It holds up to 15 kilos and is easily attached or removed. To mount the bag, stand between the handles of your rollator and apply the parking brake **A**. Then lift the bag forwards in order to slide it underneath the seat and onto the front and the carrying handles **B**.

When it is placed correctly, you will hear a click. Ensure that the brake wire still runs unrestrained along the frame. To remove the bag, stand between the handlebars and apply the parking brake. Place your hands towards the front of the bag, on either side. Lift the bag up while pressing it slightly together. This will release it. Always use both hands when performing this operation **C**.

The bag can be washed gently at 40 °C.

Edge guard // Tilt function FIG. 10

Edge guard: The edge guards **A**, which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc.

Tilt function: To use the tilt function use the front of the foot and press down on the tilt pedal **B**, which is beside the rear wheel. This helps to tilt the rollator over thresholds, edges etc. Please note: Take care when descending kerbs. Do not use the rollator on stairs.

Carrying the rollator FIG. 11

The rollator must not be carried by the seat strap or the brake wires. Fold the rollator together as shown on the illustration. Carry the rollator by the folded seat bars **A**.

Transport

The rollator should be transported in the folded and locked position (see paragraph *Unfolding / Folding*). Ensure that the lock clamp is on, so that the rollator cannot unfold unintentionally. Be careful when loading the rollator into a vehicle, paying attention to storing the item securely.

Storage

The rollator should be stored in an upright position. If stored outdoors, it should be under cover for protection. Do not leave your rollator unattended. Do not put heavy items on top of the rollator during storage.

Material // Recycling

The rollator is made of plastic-coated aluminium pipes, plastic couplings and material are made of plastic and polyester. There are ball bearings in all wheels and forks. Most rollator parts can be recycled. Disposal of the rollator and its packaging should be in accordance with the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The rollator can be steamed, hosed and washed with normal household detergents. Do not use scrub detergents, steam, or high-pressure cleaners.

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface of the rollator must be cleaned with a disinfectant containing 70–80 % ethanol. We advise against disinfectant that contains chlorine and phenol.

The manufacturer cannot be held responsible for any damage or injuries that might occur by the use of harmful disinfectant or disinfection carried out by unauthorized personnel.

Inspection // Maintenance // Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis, and at least once a year (frequency is determined by how often the rollator is used): Tubes, frame and screws, handlebars, handlebar tubes, brakes and brake handles, blocks and springs, wheels, seat and accessories. This also applies when the rollator is to be prepared for re-use. Please refer to the detailed instructions presented in this user manual about maintenance of the brakes and cleaning and disinfection of the rollator.

If the rollator does not work properly

Make sure the rollator is unfolded correctly and that both brake handles are in the appropriate position.

Accessories

For pictures and TOPRO article numbers please see the inside of the cover of this user manual.

Accessories mounted on the rollator may influence stability, we therefore advise you to use them carefully.

- | | |
|---|---|
| A Back support with padding | J Lock |
| B Tray (<i>NB: not for size S</i>) | K Umbrella with attachment (black) |
| C Anti-slip mat for tray | L Drag brake (pair) |
| D Rear bag with zipper | M Off-road wheels for IBS, set of four |
| E Transport bag | N Studded tyres for IBS, pair of two rear wheels |
| F Crutch holder | O Ice spurs for shoes |
| G Bottle holder | P One-handed brake |
| H LED lamp with clip-on and USB plug | Q Guiding handle for the visually impaired |
| I Name tag | R Basket for oxygen bottle (only compressed, not liquid) |

The accessories are to be ordered separately to enable a TOPRO OLYMPOS ATR to be customised to the needs of an individual user. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories or visit our homepage www.topromobility.com.

In case of questions or product faults, please contact your mobility shop or dealer.

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TOPRO Mobility B.V.
TOPRO Limited
Strack AG