**TOPRO
TROJA WALKER²**

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and manufactured in Norway. Due to advanced manufacturing by using the best equipment, we are able to offer great accuracy and maximum quality in all levels of our production. This product conforms to the Medical Device Directive 93/42/EEC. It is tested and approved according to EN-ISO 11199-3:2005.

On receipt of the goods, please check your product, if you have any queries, please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topromobility.com.

Warranty // Expected life span of the product

TOPRO TROJA WALKER² is guaranteed free of faults and defects for a period of 7 years.

Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake cable, wheel, seat, basket and handgrip) are exempt from the 7-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product.

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

■ The red numbered squares below refer to the numbered illustrations on the inside of the cover.

Get to know your TOPRO PRODUCT ■

- | | | |
|------------------------------------|------------------------------------|-----------------------------------|
| A Forearm support | F Seat | K Side frame |
| B Brake handle | G Lock clamp | L Brake block / Edge guard |
| C Forearm support tube | H Strap for folding | M Tilt function |
| D Brake wire | I Lock wheel for the handle | N Supporting cord |
| E Seat bars/carrying handle | J Basket | |
- O** The CE label on the rear side of the lower left tube shows the product name/model, serial number (SN), date of production and ■ Maximum weight of user ■ Maximum width of walker ■ Maximum height of walker

Measurements – TOPRO Troja Walker²

Size	M (Medium)	S (Small)	X (Extra small)
Recommended user height	150–185 cm	130–170 cm	130–160 cm
Maximum weight of user	135 kg	125 kg	80 kg
Maximum load basket	5 kg / 20 l	5 kg / 20 l	5 kg / 11 l
Height of support points	980–1150 mm	840–1000 mm	840–960 mm
Diameter hand grip	35 mm	35 mm	35 mm
Grip distance driving brake	74 mm	74 mm	74 mm
Width between support points	450 mm	450 mm	350 mm
Seating width	455 mm	455 mm	360 mm
Seat height	630 mm	550 mm	520 mm
Diameter turning circle	840 mm	840 mm	790 mm
Maximum height	1250 mm	1100 mm	1060 mm
Maximum width	630 mm	630 mm	530 mm
Maximum length	650 mm	650 mm	650 mm
Diameter front-/rear wheels	200 mm	200 mm	200 mm
Width front-/rear wheels	36 mm	36 mm	36 mm
Height when folded	1020 mm	880 mm	880 mm
Width when folded	330 mm	330 mm	330 mm
Length when folded	650 mm	650 mm	650 mm
Product weight	8.9 kg	8.6 kg	8.3 kg

Intention

The TOPRO TROJA WALKER² shall support users with poor balance and/or poor mobility. The integrated seat provides an opportunity for the user to rest if needed. The basket is designed for carrying items weighing up to 5 kg. The intended user is an adult. Height and weight restrictions apply, see table above.

Contraindications: The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities. The walker is designed and approved for indoor use on plain foundation. It is designed to be pushed, not pulled.

Important to know about safety before use

- Take care when folding and unfolding, do not trap your fingers.
- Be careful not to put your fingers in open holes or between stationary parts.
- Check the brakes every time before use and that the walker locks in the unfolded position.
- The walker is not to be used by persons heavier than the maximum user weight (see table).
- Do not use to transport heavy loads or persons.
- The walker is not a toy and therefore should be treated in an appropriate manner.
- Make sure that the lock wheel for height adjustment for the handles is locked tightly.
- Be aware that some parts of the walker might feel hot or cold when exposed to extreme temperatures.
- Training by a skilled person is advised, to learn how to use the product correctly.
- Do not alter the product as this might put your safety at risk and the guarantee will become void.

Important to know about safety when in use **2**

- When stationary or parked, always use the parking brakes on both sides **A**.
- Before sitting on the seat, make sure the parking brakes on both sides are in the locked position **A**.
- To avoid the risk of falling, do not lean or bend sideways or over the back of the walker whilst in the seated position. Do not tilt the walker, to avoid risk of falling. For the same reason, bags etc. must never be hung on the handlebars.
- The driving brakes can be held in a little **B** to reduce speed but must not be in locked position when the walker is in motion.
- To give the right support and safety, make sure the walker is kept as close to your body as possible **C**.
- Walk straight, look forward and hold your backside in **D**. This is the easiest and safest way to keep your balance.
- Do not push the walker too far from your body as this can cause unnecessary strain and you could fall.
- Do not use the walker on stairs. Be careful when passing thresholds etc.
- Be careful when wearing flared skirts and trousers as they could get caught between the brake block and wheel, which could lead to falling. We therefore recommend not wearing flared/wide clothing. Wear sturdy footwear to avoid injuries.
- For safety reasons it is prohibited to sit on the walker during public transportation. Store your walker securely and use the parking brake.
- Keep the walker away from fire and hot items.

Assembling / adjusting

TOPRO TROJA WALKER² is delivered assembled. Before use, you must unfold the walker. The height of the forearm support tubes will then need adjusting to suit the user's needs. Please read the following paragraphs carefully.

Unfolding / folding **3**

Unfolding: Loosen the strap between the forearm supports **A** (not applicable for the walker in size X). Release the lock clamp positioned at the seat bars **B**. Press the seat bars down and out **C**. The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The walker is now locked in its open position. Fasten the strap **A**.

Folding: Loosen the strap between the forearm supports **A**. Pull the strap on the seat **D** and press the side frames together. Lock the clamp **E** over the seat bars.

Height adjustment of the forearm supports and adjusting the handles **4**

Height adjustment: Unscrew the lock wheel **C** (approx. 2 turns). Adjust the forearm support tubes to a suitable height and then pull them slightly up and down **D** until you hear a clicking sound. Tighten the lock **C** again. Adjust both tubes to the same height.

Lowering: Unscrew the lock wheel **C** (approx. 2 turns). Push the forearm support tube down until it stops. Tighten the lock wheel **C** again.

Recommended height: **2**

Stand between the rear wheels. Relax your shoulders and hold your elbows in a 90-degree angle. The level of the forearm supports should be fixed at the level of your elbows. A skilled person should help you with the adjustment of the arm supports and the handles, to help avoiding bad posture.

Adjusting the handles: Unscrew the lock wheel **A**. You can now pull the handle **B** backwards or push it forward. It is also possible to adjust the angle of the handle. After adjusting the handle to the wanted position, tighten the lock wheel again. *Be careful when unscrewing/tightening the lock wheel so that you do not hurt your fingers.*

Height adjustment using the TOPRO Memory Function **5**

With the tool free TOPRO Memory Function you have the facility to set the height of the forearm supports at a standard setting to suit your requirements. This will enable you to easily find the same height again. This function may be necessary when the walker is being transported or stored as compact as possible. Decide on the height of the forearm support tube. Count visible holes at the back of the tubes. Unscrew the lock wheel **A**. Pull the tube **B** out of

the frame. The fastening piece **C** will now follow the tube. Move the fastening piece **C** on the tube **B** so the number of holes corresponds to the number counted earlier. Fit the memory chip **D** on the tube in the first available hole under the fastening piece **C**. Push the tube **B** down and into the side frame **E** until the fastening piece is locked. Screw the lock wheel **A** half in. Pull the tube **B** up until it stops, and then tighten the lock wheel **A** properly. From now on, when the forearm support tubes are in the lowest position, they will stop in the pre-set position automatically when pulled up.

Adjusting / checking / maintaining the brakes **6**

Adjusting / checking: Loosen the nut **A** beside the brake. Tighten the brake wire **B** with the adjustment screw **C**. Adjust both brake blocks so the distance to the wheel surface is approximately 1 mm **D**. Tighten the nut **A**. Make sure the brakes are not adjusted too tight and pressing into the rubber of the wheels when the parking brake is on.

Maintaining: Clean the brake block **E** and the brake spring **F** frequently. The wheels and brakes can be replaced if they are worn or damaged.

Using the brakes **2**

Make sure the brakes are working efficiently before each trip. Each brake handle works on each of the rear wheels. The walker wheels are made of a special soft rubber, which gives an optimal braking capacity on slippery floor. Wrong use can cause unsafe situations and damage to the walker: *The walker must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the brakes. This indicates incorrect use and can cause wear and tear, impairing tyres and brake quality.*

Parking brakes: Push the brake handles forward **A** to activate the parking brakes. Be aware that the handle has some resistance before it stops and locks the rear wheel. Pull the brake handles back to release the parking brakes.

Driving brakes: Pull the brake handles towards you **B** to reduce speed. *Do not use continuously while the walker is in motion.*

Seat // Basket **7**

The walker has a safe net seat, through which you have a view to the ground while walking. It can easily be wiped with a cloth if needed. The parking brakes must be on before using the seat **A**. Push both brake handles **B** forwards until they lock. Both rear wheels are now locked. Sit with your back towards the basket **C**. The basket **D** can carry goods up to 5 kg. It can be removed and washed gently at 40 °C.

Edge guard // Tilt function **8**

Edge guard: The brake blocks **A**, which are in front of the rear wheels, prevent the rear wheels from catching on corners, door frames etc. **Tilt function:** To use the tilt function use the front of the foot and press down on the tilt pedal **B**, which is beside the rear wheel. This helps to tilt the walker over thresholds, edges etc. Please note: Take care when passing obstacles such as thresholds etc. Do not use the walker on stairs.

Carrying the walker **9**

The walker must not be carried by the seat strap or the brake wires. Fold the walker together as shown on the illustration. Carry the walker by the folded seat bars **A**.

Transport

The walker should be transported in the folded and locked position (see paragraph *Unfolding / Folding*). Ensure that the walker cannot unfold unintentionally. Be careful when loading the walker into a vehicle, paying attention to storing the item securely.

Storage

The walker should be stored in an upright position. If stored outdoors, it should be under cover for protection. Do not leave your walker unattended. Do not put heavy items on top of the walker during storage.

Material // Recycling

The walker is made of plastic-coated aluminium pipes, plastic couplings and parts of plastic and polyester. Most parts can be recycled. Disposal of the walker and its packaging according to the applicable regulations should be in accordance with the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The walker can be steamed, hosed and washed with normal household detergents. Do not use scrub detergents, steam, or high-pressure cleaners.

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface must be cleaned with a disinfectant containing 70–80% ethanol. We advise against disinfectant that contains chlorine and phenol. *The manufacturer cannot be held responsible for any damage or wounds that might occur by the use of harmful disinfectant or disinfection carried out by unauthorized personnel.*

Inspection // Maintenance // Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis, and at least once a year (frequency is determined by how and how often the walker is used): Tubes, frame and screws, forearm supports and handles, brake parts, wheels, seat and accessories. This also applies when the walker is to be prepared for re-use. Please refer to the detailed instructions presented in this user manual about maintenance of the brakes and cleaning and disinfection of the walker.

If the walker does not work properly

Make sure the walker is unfolded correctly and that both brake handles are in the appropriate position.

Accessories

For pictures and TOPRO article numbers please see the inside of the cover of this user manual. Accessories mounted on the walker may influence stability, we therefore advise you to use them carefully.

- | | |
|--|--|
| A Tray (NB: not for size X) | I Basket for oxygen bottle (only compressed, not liquid) (NB: not for size X) |
| B Anti-slip mat for tray (NB: not for size X) | J Name tag |
| C Crutch holder | K Lock |
| D Back support | L One-handed brake |
| E Stabilizer bar | M Guiding handle for e.g. visually impaired |
| F Rear bag with zipper (NB: not for size X) | N Drag brake (pair) |
| G Bottle holder | |
| H Flower stickers, set of four (NB: not for size X) | |

The accessories are to be ordered separately to give the possibility to enable a TOPRO TROJA WALKER² to be customised to the needs of an individual user. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topromobility.com.

In case of questions or product faults please contact your mobility shop or dealer.

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TOPRO Mobility B.V.
TOPRO Limited