

Eloflex A-KIT - Mounting instructions

November 2025



To facilitate maneuvering the wheelchairs from behind (as assistant, companion etc.) the wheelchairs can be equipped with an A-kit. It consist of an Assistant handle and a thumb joystick.



1. Assemble the handles by tightening the two screws according to the picture.



2. Adjust the height of the handle by loosening the screw according to the picture.



The control unit can be mounted on either the left or right side.

Remove the control unit, reposition the parts in the order shown in the pictures, (1,2,3) then reattach the control unit.



3. Fasten the thumb joystick on the mounting bar.



4. Connect the cable from the control unit to the joystick according to the picture.

Eloflex A-KIT - Mounting instructions

November 2025



Attendant control functions.

The control unit is operated using the ON/OFF button (1) and the thumb joystick (2).

- To turn on the system, press the ON/OFF button (1).
- To change the speed, press the ON/OFF button (1) briefly. The speed increases one level with each press and cycles back to the lowest speed after the highest level. The current speed is shown by the LEDs on the main joystick in five steps.
- To drive the wheelchair, move the thumb joystick (2).
- To turn off the system, press and hold the ON/OFF button (1) for 3 seconds.

Eloflex A-KIT - Mounting instructions Model C3


November 2025



To facilitate maneuvering the wheelchairs from behind (as assistant, companion etc.) the wheelchairs can be equipped with an A-kit. It consist of an Assistant handle and a thumb joystick.



1. Remove the fastening clamps completely from the handle frame. From behind, insert the handle frame into the top of the backrest chassis.

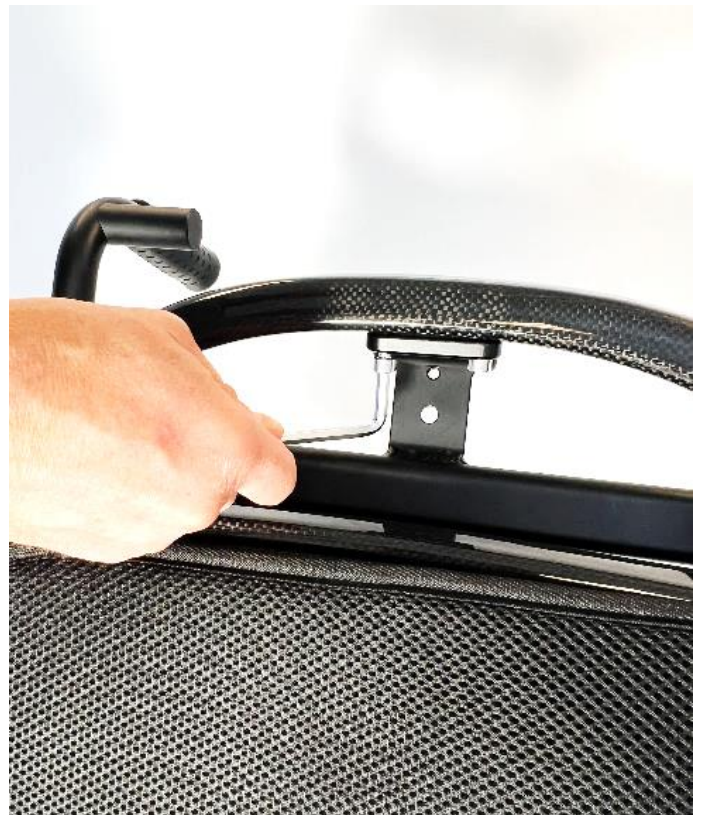

2. Align the top bracket of the handle frame with the two holes on the bottom of the to backrest bar. Fasten the the screws firmly with a hex key. **NOTE:** Be careful to not over tighten due to the nature of carbon fiber material.



3. Mount and tighten (A) the left and right fastening clamps. Adjust desired height (B) of the handles and fix with the predefined positions/holes and bolts.



For the left joystick position, adjust as follows:
A. Remove the adjustable handles and switch them left to right.
B. The mounting bar will now face outward. Loosen the screw, rotate the bar so it points inward, and re-mount it.



4. Fasten the thumb joystick on the mounting bar.



3. Route the joystick cable in a practical way to avoid getting caught or damaged while using or folding the wheelchair. Connect it to the charging port of the main joystick.



Attendant control functions.

The control unit is operated using the ON/OFF button (1) and the thumb joystick (2).

- To turn on the system, press the ON/OFF button (1).
- To change the speed, press the ON/OFF button (1) briefly. The speed increases one level with each press and cycles back to the lowest speed after the highest level. The current speed is shown by the LEDs on the main joystick in five steps.
- To drive the wheelchair, move the thumb joystick (2).
- To turn off the system, press and hold the ON/OFF button (1) for 3 seconds.