


# Headrest - Mounting instructions

November 2025

The headrest can be adjusted in most directions and comes with a quick lock function for transport removal.



1. Firmly fasten one side of the headrest onto the backrest by using the mounting clamp.

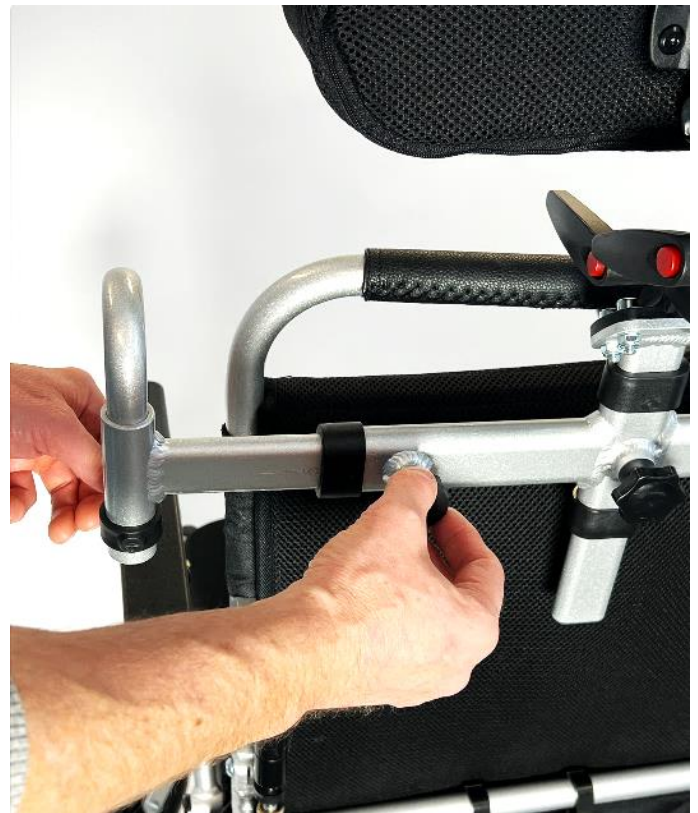

2. Adjust the width of the headrest frame (if needed) by opening the horizontal adjustment knob.



3. Make sure the assembly is straight and level, then tighten the mounting clamps firmly.



4. If desired the base position of the headrest can be now adjusted sideways or up/down with the 3 adjustment knobs. Tighten firmly.

# Headrest - Mounting instructions

November 2025



5. The quick lock pins can be used to remove the headrest during transport.



6. Slide the headrest frame downwards.

## Model H

When mounting a headrest onto Model H an additional/replacement clamp is required:

Clamp 19mm Model H (7350006084195)

# Headrest C3 - Mounting instructions

November 2025

The C3 headrest can be mounted directly on the C3 backrest, or onto the C3 assistant handles.



1. Place the mounting bracket onto the backrest from behind and align with the holes. Tighten the two screws with a hex key.  
**NOTE:** Be careful not to overtighten due to the nature of carbon fiber material.



2. Slide the headrest onto the bracket from above.




3. The central locking knob is used to fix the headrest at the desired height, and to remove the headrest during transport if needed.



4. Fine-tune the position (front/back, up/down) of the headrest with the three levers at the back.

# Headrest C3 - Mounting instructions

November 2025

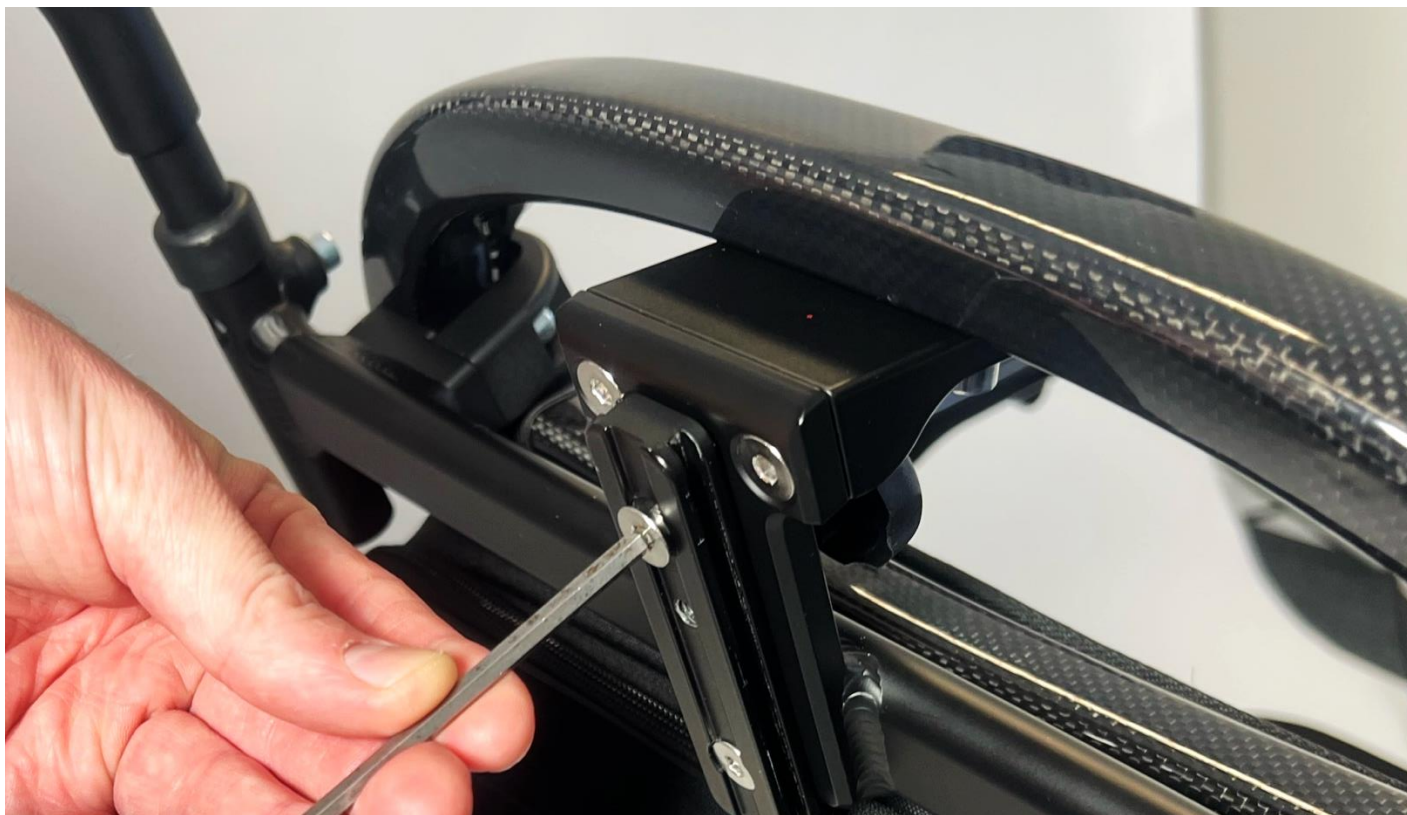

5. If combined with an assistant handle, move the height-adjustment "rail track" onto the assistant-handle frame. Take care not to mix up the up/down direction so the central hole aligns.