

Widening Kit – Mounting instructions Model F, L, P, D2

November 2025


EL&FLEX

Eloflex widening kit are used to increase the distance between the armrests of the wheelchair for a more comfortable seating space.

The extra widths comes as +5, +7 or +10cm depending on model.



1. Remove the armrest by unscrewing the the upper and lower bolt with a hex wrench (Allen key).

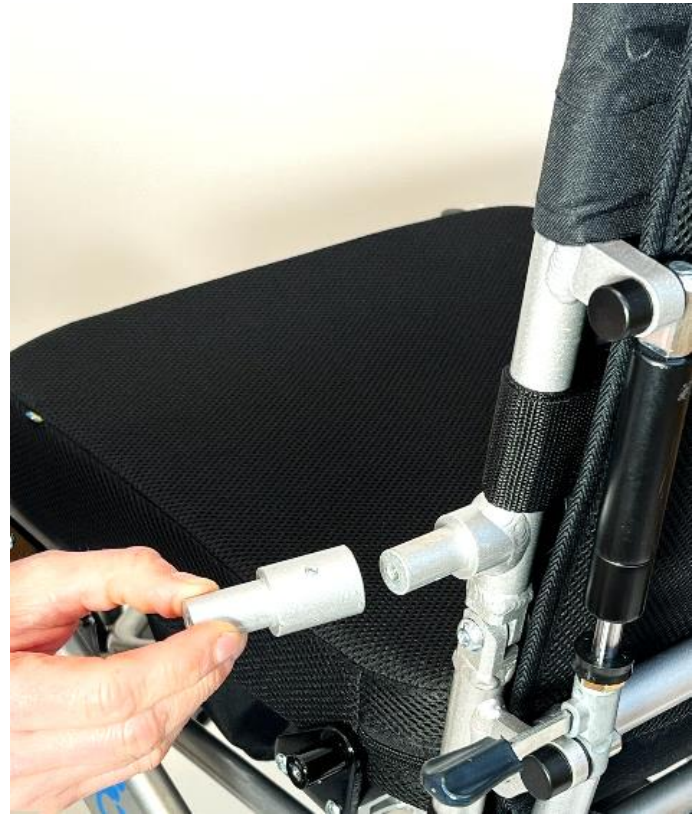

2. Mount the tube spacers onto the chassis. Please note that there are two differ lengths of spacers that will match corresponding sockets.



3. Securely tighten the small hex screws that will hold the spacers in place.



4. Reassembly the armrest again onto the spacers using the original bolts. After 1-2 weeks, please check the small hex screws and retighten them if needed.

Widening Kit – Mounting instructions Model H

November 2025

Eloflex widening kit are used to increase the distance between the armrests of the wheelchair for a more comfortable seating space.


The extra widths comes as +5, +7 or +10cm depending on model.



1. Remove the armrest completely by loosening the quick lock adjustment and pulling it upward.



2. Undo the bolts holding the armrest clamp using a ring spanner and a hex wrench (Allen key) and remove the clamp.

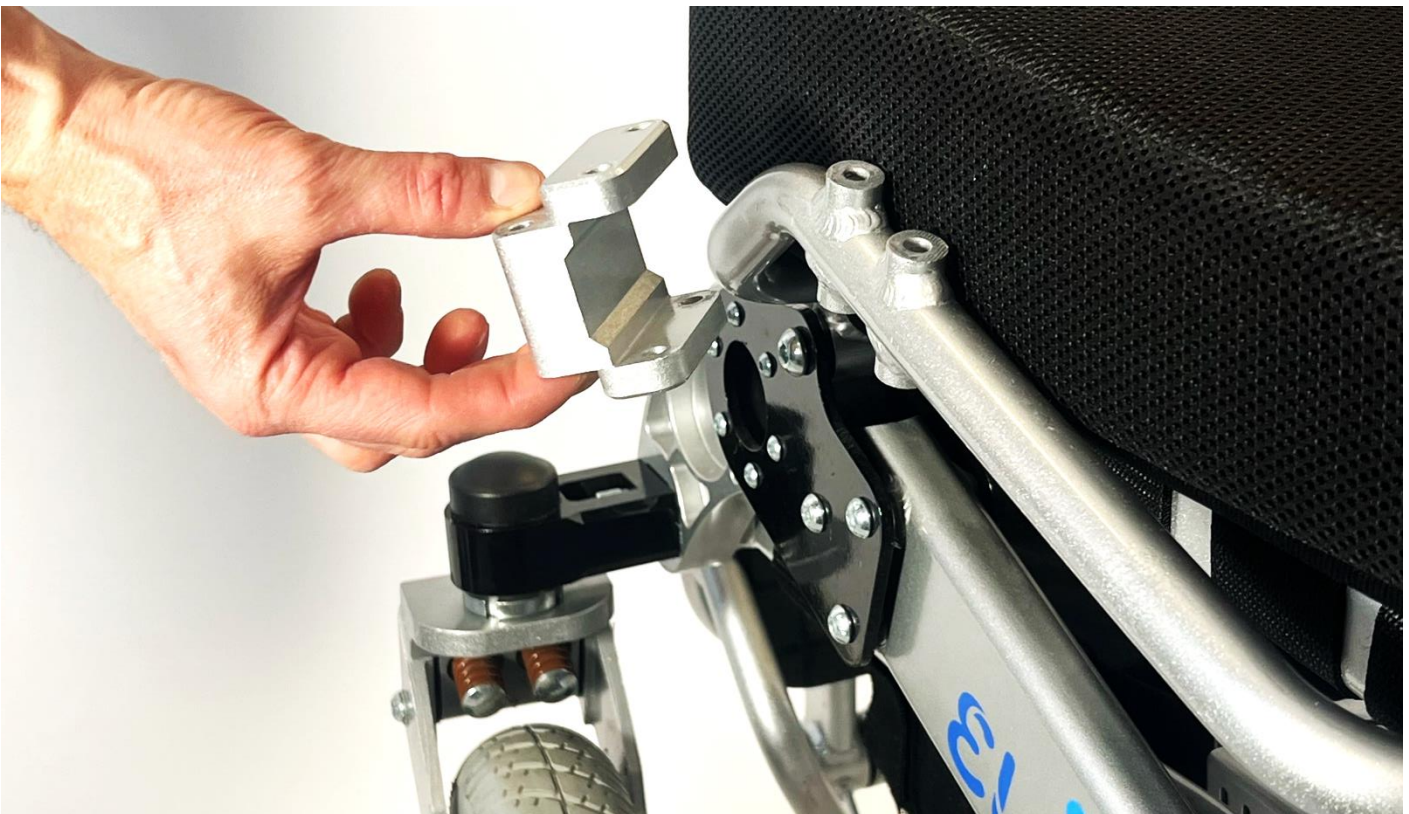

3. Place the spacer block onto the bolt holes and reassembly the armrest again as normal.

Widening Kit – Mounting instructions Model C3

November 2025

EL&FLEX

Eloflex widening kit are used to increase the distance between the armrests of the wheelchair for a more comfortable seating space.

The extra widths comes as +5, +7 or +10cm depending on model.



1. Remove the armrest by unscrew the central bolt with a hex wrench (Allen key).



2. Mount the 4 pcs inner tap spacers onto the chassis, securing them tightly with an open-ended spanner (wrench).



3. Slide all the outer tube spacers in place



4. Reassembly the armrest again as normal.

Widening Kit – Mounting instructions Model R

November 2025

EL&FLEX

Eloflex widening kit are used to increase the distance between the armrests of the wheelchair for a more comfortable seating space.

The extra widths comes as +5, +7 or +10cm depending on model.




1. Remove the two bolts holding the lower bracket plate with a hex wrench (Allen key).



2. Remove the upper armrest mount by unscrewing the central bolt and remove the armrest.



3. Slide the seat spacer onto the mounting bracket

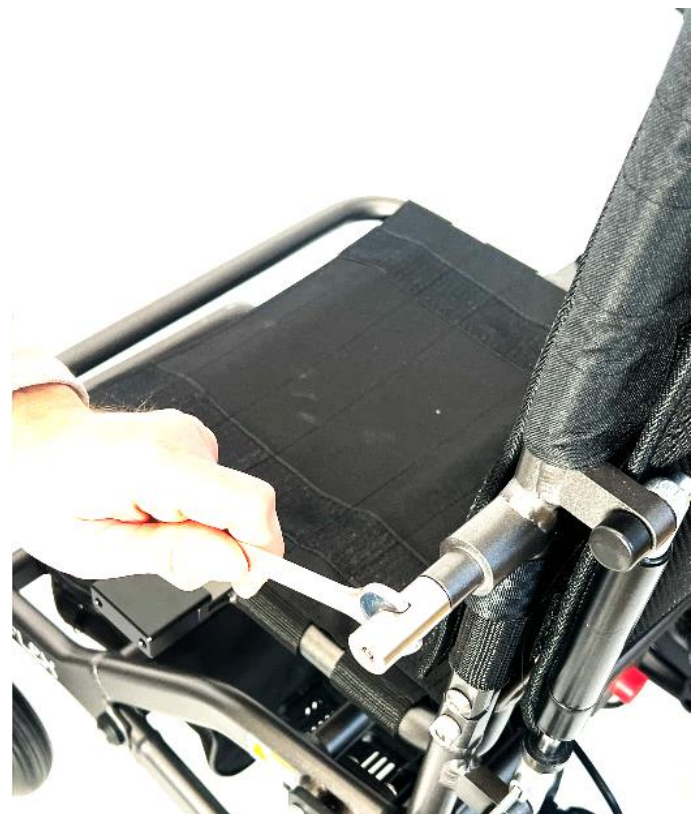

4. Mount the inner tap spacer onto the backrest chassis, securing them tightly with an open-ended spanner (wrench).

Widening Kit – Mounting instructions Model R

November 2025

EL&FLEX



5. Slide the outer tube spacer in place



6. Put the armrest back in place and secure it with the central bolt onto the backrest.



7. Secure the seat spacer and bracket with the 2 longer bolts onto the seat chassis.