
- Ⓚ **Schwellenrampen**
- ⓐ **Doorstep ramps**
- ⓓ **Dørtrinsramper**
- Ⓦ **Rampe da soglia**

Art.Nr. **DS4025**
Item no. **DS4025**
Varenr. **DS4076**
Codice **DS4076**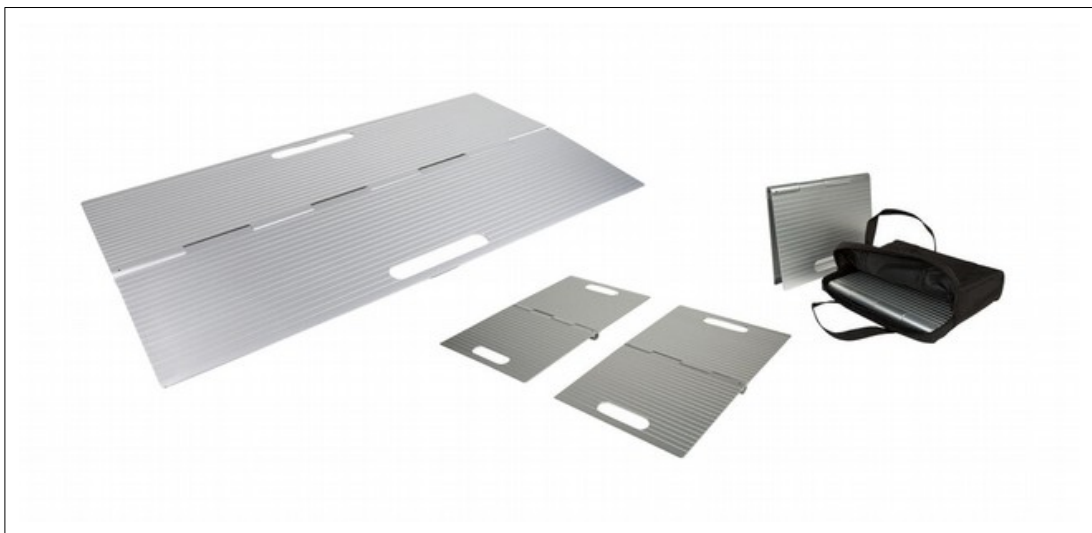

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Danimarca
www.mobilex.dk

D Schwellenrampe DS4025/DS4070

Produkt Beschreibung

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GB Doorstep ramp DS4025/DS4076

Product description

Mobile and height-adjustable threshold ramp from aluminium with anti-skid surface. The supplied feet are adjustable in height.

Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose and only on even and flat surfaces.

Shipment

The DS ramps are delivered in a box. The model DS4025 also comes with a carrying bag.

Assembly and use

The DS ramps consist of two parts which are connected by a hinge and are delivered already assembled. The model DS4070 consists of a single wide part, while the model DS4025 consists of two separate narrower parts.

For use put the ramp over the obstacle to be crossed (max. height 5 cm) so that the two sides with the milled handles are on the opposite side of the obstacle.

Cleaning & maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning (no solvents or abrasives).

Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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